# G3FERRARI®

BY TREVIDEA

MOD.:  
**G30054**

**MANUALE D'USO**  
**USER MANUAL**

IT

EN

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## **ENGLISH: SAFETY INSTRUCTIONS**

- Read all information in operation guide and any other literature in the box before operating the unit.
- The appliance is designed only for its intended use. Do not misuse in any way.
- The instruction manual is an important part of the product, and contains important information about the function and use of the device.
- Read all the information on giftbox and instruction manual before using the device.
- Preserve the instruction manual for future reading. If the device is given to other persons, the instruction manual must be given together with the device.
- The patient is an intended operator.
- Please do not use other cuff than the one supplied by the manufacturer, otherwise it may bring biocompatible hazard and might result in measurement error.
- Please do not share the cuff with other infective person to avoid cross-infection.
- Babies, young children or individuals who cannot express their consent are not suitable to take blood pressure measurement.
- The monitor might not meet its performance specifications or cause safety hazard if stored or used outside the specified temperature and humidity ranges in specifications.
- Do not use the device near to strong electrical or electromagnetic fields generated by cell phones or other, they may cause incorrect readings and interference or become interference source to the device.
- Do not disassemble the unit or cuff. Do not attempt to repair.
- The cuff should not be applied over a wound as this can cause further injury.
- DO NOT attach the cuff to a limb being used for IV infusions or any other intravascular access, therapy or an arterio-venous (A-V) shunt. The cuff inflation can temporarily block blood flow, potentially causing harm to the patient.
- Individuals with serious arteriosclerosis or arrhythmia are not suitable to take blood pressure measure.
- Individuals with serious circulation problems may experience discomfort. Consult your physician

prior to use.

- **Do not confuse self-monitoring with self-diagnosis. If there are unusual values, consult your doctor.** Blood pressure measurements should only be interpreted by a health professional who is familiar with your medical history.
- Do not attempt self-treatment without consulting your physician first.
- Consult your physician if
  - 1) you have doubt about the application of the cuff over a wound or inflammation diseases;
  - 2) you have doubt about the application of the cuff on any limb where intravascular access or therapy, or an arterio-venous (A-V) shunt, is present;
  - 3) you have doubt about the application of the cuff on the arm on the side of a mastectomy;
  - 4) you are under medical treatment, to determine the moment of the day in which is more correct to take the measurement of the blood pressure.
  - 5) you suffer of atrial or ventricular arrhythmias, vascular constriction, liver disorders or diabetes, or if you have cardiac pacemakers or a weak pulse, or if you are pregnant: these factors may produce incorrect readings
  - 6) Simultaneously used with other monitoring medical equipment on the same limb.
  - 7) Need to check the blood circulation of the user.
- If Irregular Heartbeat (IHB) brought by common arrhythmias is detected in the procedure of blood pressure measurement, a signal IHB will be displayed. Under this condition, the Electronic Sphygmomanometers can keep function, but the results may not be accurate, it's suggested that you consult with your physician for accurate assessment. Handling batteries properly. As soon as old batteries run out, replace with new batteries.
- Do not use old and new batteries together. Align the polarities of batteries correctly.
- When the unit will not be used for more than 3 months, remove the batteries. Otherwise, batteries may leak and cause damage to the unit.
- Swallowing batteries and/or battery fluid can be extremely dangerous. Keep the batteries and the unit out of the reach of children and disabled persons.

### **INTENDED USE**

Fully Automatic Electronic Sphygmomanometer is for use by medical professionals or at home and is a non-invasive blood pressure measurement system intended to measure the diastolic and systolic blood pressures and pulse rate of an adult individual by using a non-invasive technique in which an inflatable cuff is wrapped around the upper arm.

### **QUICK START GUIDE**

All physical activity, excitement, stress, eating, drinking, smoking, body posture and many other activities or factors (including taking a blood pressure measurement) will influence blood pressure value. Because of this, it is mostly unusual to obtain identical multiple blood pressure readings.

Blood pressure fluctuates continually day and night. The highest value usually appears in the daytime and lowest one usually at midnight. Typically, the value begins to increase at around 3:00AM, and reaches to highest level in the daytime while most people are awake and active.

Considering the above information, it is recommended that you measure your blood pressure at approximately the same time each day.

Too frequent measurements may cause injury due to blood flow interference, please always relax a minimum of 1 to 1.5 minutes between measurements to allow the blood circulation in your arm to recover. It is rare that you obtain identical blood pressure readings each time.

## **OVERLOOK G30054**

- |                                  |                        |                              |
|----------------------------------|------------------------|------------------------------|
| 1. Blood pressure classification | 2. Display LCD         | 3. Systolic blood pressure   |
| 4. Diastolic blood pressure      | 5. Pulse rate          | 6. "M" button                |
| 7. "START/STOP" button           | 8 "SET" button         | 9. Cuff                      |
| 10. Low battery indicator        | 11. Average 3 measures | 12. Irregular beat indicator |
| 13. Hour                         | 14. Date               |                              |

# PREPARATION

## BATTERY LOADING

- Open battery cover.
- Load two “AAA” size batteries. Please pay attention to polarity.
- Close the battery cover.

When LCD shows battery symbol , replace all batteries with new ones.

Rechargeable batteries are not suitable for this monitor. Remove the batteries if the monitor will not be used for a month or more to avoid relevant damage of battery leakage.

Avoid the battery fluid to get in your eyes. If it should get in your eyes, immediately rinse with plenty of clean water and contact a physician.

## SET THE USER FOR MEASURE SAVING

The device is equipped with a memory containing 120 measures.


These memories are divided in 2 groups of 60 measures each.

In case 2 persons use the appliance, you have to associate one user to the memory group 1, and the other user to the group 2.

With device off, press button SET (8), the icon of the group flashes.


Press button M (6) to select the memory group in which save the measure, then confirm with START/STOP.

**Following measures will be saved in the group set. If the person changes, remember to change user.**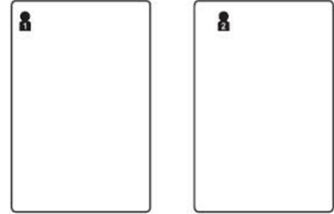

## CLOCK AND DATE ADJUSTMENT

- After setting the user, press again the button SET (8) to enter Clock setting Mode.
- The digits of the months flash. Set the month with the button M (6).
- Press SET to pass to the days and set with button M.
- Repeat operation for hours and minutes.
- Confirm settings by pressing START/STOP (7).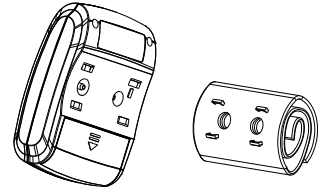


## CONNECTING THE CUFF TO THE MONITOR

The cuff is attached to the monitor when it is packaged. If cuff become unattached, align the two plugs and four brackets of the cuff with the plug sockets and bracket sockets of the monitor and press the cuff to the monitor until the plugs and brackets are securely attached.

## WRIST CUFF and BODY POSTURE

Place the cuff around a bare wrist 1-2cm above the wrist joint on the palm side of the wrist. While seated, place the arm with the cuffed wrist in front of your body on a desk or table with the palm up. If the cuff is correctly placed, you can read the LCD display. The cuff must be neither too tight nor too loose.

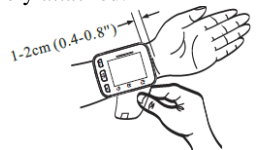### NOTE:

- Always measure the same wrist.
- Do not move arm, body or monitor during measure.
- Stay still and quiet for 5 minutes before measuring.
- Wait 3 minutes between 2 measurements.
- Be seated with your feet flat on the floor, and don't cross your legs. Place palm upside in front of you on a flat surface such as a desk or table. The middle of the cuff should be at the level of the right atrium of the heart.
- Keep the cuff clean. If the cuff becomes dirty, remove it from the monitor and clear it by hand in a mild detergent, then rinse it thoroughly in cold water. Never dry the cuff in clothes dryer or iron it. Clean the cuff after the usage of every 200 times is recommended.



## MEASURING

After applying the cuff and your body is in a comfortable position, press the “START/STOP” button until you hear a beep. All display characters are shown for self-test. See picture.

NOTE: Unit will not function if residual air from previous testing is present in cuff.

The LCD will flash  until pressure is stabilized.

- The monitor inflates the cuff until sufficient pressure has built up for a measurement. Then the monitor slowly releases air from the cuff and carries out the measurement. Keep relaxed during testing. Avoid speaking or moving body parts.
- Finally the blood pressure and pulse rate will be calculated and displayed on the LCD screen separately. In case of irregular heartbeat the symbol (IHB) will blink.. The result will be automatically stored in the memory bank.
- If you want to cancel the measure because wrong, press the "SET" button after result is displayed.
- After measurement, the monitor will turn off automatically after 3 minutes of no operation. Alternatively, you can press the “START/STOP” button to turn off the monitor manually.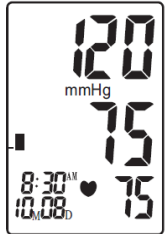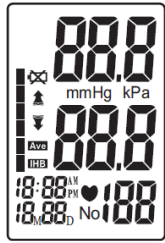

Note: Please consult a health care professional for interpretation of pressure measurements.



## DISPLAYING STORED RESULTS

You can review the results in the memory bank by pressing the “M”

button (6). The data of the set group/user are shown (♂ or ♀).

With device off, press button “M” (6).

- At the first pressing, the LCD displays the average of last 3 tests (indicated by “Ave” (11).
- At the second pressing the LCD displays the last measurement (01).
- Pressing again the M button the LCD displays previous measurements (02, 03, 04, ecc).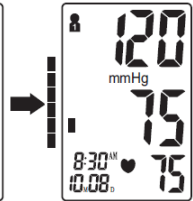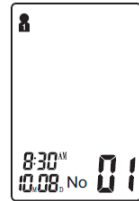


## DELETING MEASUREMENTS FROM THE MEMORY

To cancel the memory (**WARNING: all the memories of a group, not only one**), press the button SET for 3 seconds. Cancelling only one measure is possible only at the end of the measure itself, as described in previous paragraph.

## ASSESSING HIGH BLOOD PRESSURE FOR ADULTS

The following guidelines for assessing high blood pressure (without regard to age or gender) have been established by the World Health Organization (WHO). Please note that other-factors (e.g. diabetes, obesity, smoking, etc.) need to be taken into consideration. Consult with your physician for accurate assessment, and never change your treatment by yourself.

### Classification of blood pressure for adults

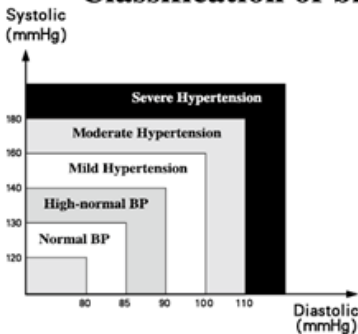BLOOD PRESSURE CLASSIFICATION	SBP mmHg	DBP mmHg
Optimal	<120	<80
Normal	120-129	80-84
High-Normal	130-139	85-89
Grade 1 Hypertension	140-159	90-99
Grade 2 Hypertension	160-179	100-109
Grade 3 Hypertension	≥ 180	≥ 110

WHO/ISH Definitions and Classification of Blood Pressure Levels

## TROUBLESHOOTING (1)

PROBLEM	POSSIBLE CAUSE	SOLUTION
LCD Display shows abnormal result	The cuff position was not correct or it was not properly tightened	Apply the cuff correctly and try again
	Body posture was not correct during testing	Review the “WRIST CUFF and BODY POSTURE” sections of the instructions and re-test.
	Speaking, arm or body movement, angry, excited or nervous during testing	Re-test when calm and without speaking or moving during the test
	Irregular heartbeat (arrhythmia)	It is inappropriate for people with serious arrhythmia to use this Electronic Sphygmomanometer.

## TROUBLESHOOTING (2)

PROBLEM	POSSIBLE CAUSE	SOLUTION
LCD shows low battery symbol 	Low Battery	Change the batteries
Appare “Err”	Cuff fails to inflate properly	Make sure hose is properly fastened to cuff and monitor unit
Appare “Err”	Improper operation	Read user manual carefully and re-test properly.
Appare “Err”	Pressurization is over cuff rated pressure 300mmHg	Read user manual carefully and re-test properly.
No response when you press button or load battery.	Incorrect operation or strong electromagnetic interference.	Take out batteries for 5 min, then reinstall them.

## ELECTROMAGNETIC COMPATIBILITY INFORMATION

Table 1 - Emission

Phenomenon	Compliance	Electromagnetic environment
Radiated emissions CISPR 11	Group 1, Class B	Home healthcare environment
Conducted emissions CISPR 11	N/A	-
Harmonic emissions IEC 61000-3-2	N/A	-
Voltage fluctuations and flicker IEC 61000-3-3	N/A	-

Table 2 – Immunity

Phenomenon	Basic EMC standard	Immunity test levels
		Home Healthcare Environment
Electrostatic Discharge (ESD)	IEC 61000-4-2	±8 kV contact ±2kV, ±4kV, ±8kV, ±15kV air
Rated power frequency magnetic fields 50/60Hz	IEC 61000-4-8	30A/m 50Hz or 60Hz
Radiated RF	IEC 61000-4-3	385MHz, 27 V/m

		450MHz, 28 V/m 710, 745, 780MHz, 9 V/m 810, 870, 930MHz, 28 V/m 1720, 1845, 1970MHz, 28 V/m 2450MHz, 28 V/m 5240, 5500, 5785MHz, 9 V/m
--	--	---

## STORAGE AND MAINTENANCE

1. Do not drop this monitor or subject it to strong impact.
2. Avoid high temperature and solarization. Do not immerse the monitor in water as this will result in damage to the monitor. If this monitor is stored near freezing, allow it to acclimate to room temperature before use (around 30 minutes).
3. If you do not use the monitor for a long time, please remove the batteries.
4. It is recommended the performance should be checked every 2 years or after repair. Please contact the service center. Do not attempt to disassemble this monitor.
5. Clean the monitor with a dry, soft cloth or a soft cloth squeezed well after moistened with water, diluted disinfectant alcohol, or diluted detergent.
6. It is recommended the cuff should be disinfected 2 times every week if needed (For example, in hospital or in clinique). Wipe the inner side (the side contacts skin) of the cuff by a soft cloth squeezed after moistened with Ethyl alcohol (75-90%), then dry the cuff by airing.
7. Not servicing/maintenance while the monitor is in use.

## SPECIFICATIONS

- Model G3Ferrari G30054
- Measurement Method oscillometric, inflation Pump driven
- Cuff diameter: 13-21.5cm
- Measurement Range
  - o Cuff pressure: 0-300mmHg
  - o Systolic: 60-260mmHg
  - o Diastolic: 40-199mmHg
  - o Pulse rate: 40-180 battiti/minuto
- Accuracy: Pressure: +/- 3 mmHg, Pulse rate: +/- 5%
- Power supply 2xAAA batteries
- Environment temperature for operation: 10°C~40°C (50°F~104°F)
- Environment humidity for operation: 15-93% RH
- Environment temperature for storage and transport: -25°C~70°C (-13°F~158°F)
- Environment humidity for storage and transport: ≤93% RH
- Environment pressure: 700hPa-1060hPa
- Battery life: Approx 2 months at 3 tests per day

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## **WASTE OF ELECTRIC AND ELECTRONIC EQUIPMENT**

The product is made of non-biodegradable and potentially polluting substances if not properly disposed of; other parts can be recycled. It's our duty to contribute to the ecological health of the environment following the correct procedures for disposal. The crossed out wheeled bin symbol indicates the product complies with the requirements of the new directives introduced to protect the environment (2002/96/EC, 2003/108/EC, 2002/95/EC, 2012/19/EC) and must be properly disposed at the end of its lifetime. If you need further information, ask for the dedicated waste disposal areas at your place of residence. Who does not dispose of the product as specified in this section shall be liable according to the law.

## **INFORMATION ON THE DISPOSAL OF BATTERIES**

Whatever the type of exhausted battery, it must not be thrown in the household waste, in the fire or in the water. Each type of exhausted battery must be recycled or disposed of in the appropriate dedicated collection centers.

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*To find the nearest service center, please call the telephone number 0541 694246, fax number 0541 756430 or contact us at [assistenza@trevidea.it](mailto:assistenza@trevidea.it) asking for the service centers supervisor.*

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*Appliance is conform to the pertinent European regulation and is therefore marked with the CE mark.*

**CE 0123**

[www.g3ferrari.it](http://www.g3ferrari.it)

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**TREVIDEA S.R.L. – Strada Consolare Rimini – San Marino n. 62 – 47924 – RIMINI  
(RN) – ITALY**

**Joytech Healthcare Co., Ltd.**

**No. 365, Wuzhou Road, 311100 Hangzhou, Zhejiang Province, People's Republic of China.**

**Shanghai International Holding Corp. GmbH (Europe)  
Eiffestrasse 80, 20537 Hamburg, Germany.**

MADE IN CHINA

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**TREVIDEA S.r.l. si scusa inoltre per eventuali errori di stampa.**