G3FERRARI®

BY TREVIDEA

MOD.:
G30056

MANUALE D'USO
USER MANUAL

IT

EN

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ENGLISH

INTENDED USE

This device is an AC-powered air nebulizer compressor system intended to provide a source of compressed air for medical purposes for use in home healthcare. It is to be used with the kit nebulizer to produce aerosol particles of medication for respiratory therapy for both children and adults. The compressor nebulizer system is for aerosol therapy. And it is supplied with the kit nebulizer.

SAFETY INSTRUCTIONS

The instruction manual is an important part of the product and contains important information about the function and use of the device. Read all the information on giftbox and instruction manual before using the device and preserve the instruction manual for future reading. If the device is given to other persons, the instruction manual must be given together with the device. The following are important notes on the installation, use and maintenance; save this instruction manual for future reference; use the equipment only as specified in this guide; any other use is considered improper and dangerous; therefore, the manufacturer cannot be held responsible in the event of damages caused by improper, incorrect or unreasonable use.

Before use, ensure the equipment is undamaged; if in doubt, do not attempt to use it and contact the authorized service center; do not leave packing materials (i.e. plastic bags, polystyrene foam, nails, staples, etc.) within the reach of children as they are potential sources of danger; always remember that they must be separately collected.

Make sure that the rating information given on the technical label are compatible with those of the electricity grid; the installation must be performed according to the manufacturer's instructions considering the maximum power of the appliance as shown on the label; an incorrect installation may cause damage to people, animals or things, for which the manufacturer cannot be considered responsible thereof.

If it is necessary to use adapters, multiple sockets or electrical extensions, use only those that comply with current safety standards; in any case do not exceed the power consumption limits indicated on the electrical adapter and / or extensions, as well as the maximum power shown on multiple adaptor.

Do not leave the unit plugged; better to remove the plug from the mains when the device is not in use. Always disconnect from power supply if you leave it unattended.

Cleaning operations should be carried out after unplugging the unit.

If the unit is out of order and you do not want to fix it, it must be made inoperable by cutting power cord.

- Do not allow the power cord to get closer to sharp objects or in contact with hot surfaces; do not pull it to disconnect the plug.
- Do not use the appliance in the event of damage to the power cord, plug, or in case of short circuits; to repair the product only address an authorized service center.
- Do not handle or touch the appliance with wet hands or bare feet. Do not expose the appliance to harmful weather conditions such as rain, moisture, frost, etc.. Always store it in a dry place.
- This appliance can be used by children of 8 years and above and persons with reduced physical, sensory or mental capabilities or lack of experience and knowledge if they received supervision or instruction concerning use of the appliance in a safe way and understand hazards involved; children shall not play with the appliance; cleaning and user maintenance shall not be made by children without supervision.

- Keep appliance and its cord out of reach of children less than 8 years.

- Children must not play with the appliance.

If the power cord is damaged, or in case of failure and / or malfunction do not tamper with the unit. The repair must be done by manufacturer or by service center authorized by the manufacturer in order to prevent any risk. Failure to comply with the above may compromise the unit safety and invalidate warranty.

- This appliance is intended to be used in household and similar applications such as: staff kitchen areas in shops, offices and other working environments; by clients in hotels, motels and other residential type environments; farm houses, bed and breakfast type environments.

- The appliance is not intended to be operated by means of an external timer or separate remote-control system. Before every use, unroll the power cord.

- Use original spare parts and accessories only. Do not obstruct air ventilation holes.

- Do not subject the product to strong impacts, serious damages may result. Don't use near or under flammable materials (like curtains), heat, cold spots and steam.

USE PRECAUTIONS

- Use the device only for its intended purpose. Follow the instructions of your doctor.
- Verify that the medications prescribed by your doctor can be used with common compressor nebulizer systems for aerosol therapy.
- If the unit is not working properly, or you feel sick, stop using immediately.
- This device rated for intermittent use 30 minutes ON/ 30 minutes OFF.
- Keep away from eyes during function, as some medicines may damage eyes.
- Correct operation can be affected by electromagnetic interference that exceeds the limits indicated by the European standards. If this device interferes with another electrical apparatus, remove it and plug it into a different electrical outlet.
- Do not disassemble the unit. Do not attempt to repair.
- Use only original accessories and components (filter - accessories) to not compromise function.
- The connection tube between nebulizer and appliance must be free, not grooved and not pressed.
- After use wash and disinfect nebulizer and mask, especially if used by infective person.
- The compressor might not meet its performance specifications or cause safety hazard if stored or used outside the specified temperature and humidity ranges in specifications
- The device is unsuitable and should not be used in the following situations
 - Severe hypoxia or respiratory failure patients are forbidden use.
 - In presence of anesthetics, flammable gas or oxygen.
 - It is not used for life support or maintain.
 - Children or disabled people need to be used under adult supervision.
 - It is expected to use only in conscious patients.
 - The children under 5 years should use the mask to inhalation therapy, not the mouthpiece.

OVERLOOK G30056

1. Power switch

2. Filter

3. Tube connector

4. Air vents

5. Nebulizer support

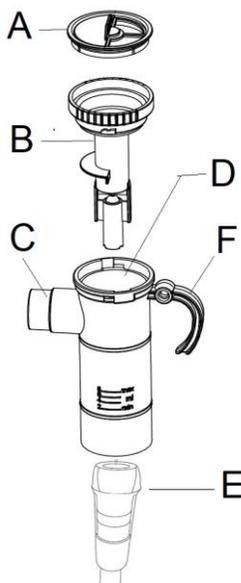
6. Nebulizer

7. Mouthpiece

8. Tube

9. Mask

10. Pediatric mask

ACCESSORIES

The machine is supplied with **nebulizer** (6).

1) Unscrew the upper part (A+B) of nebulizer.

2) Pour the medicine in the container (D). Maximum capacity is 6ml.

3) Mount the upper part (A+B) on the container (D).

4) Connect one side of the tube (8) to the air outlet (3).

5) Connect the other side of the tube (E) to the lower part of nebulizer.

On the lid (A) is present a regulation of the nebulization rate.

Open for big nebulization rate. Close for low nebulization rate.

It is possible to use one of the 3 included accessories:

- **Mask** (9), for adults.
- **Pediatric mask** (10), for children.
- **Mouthpiece** (7), if for nose condition or for type of treatment is better than the mask.

Mount these accessories to the air outlet (C) of nebulizer.

The nebulizer (6) features an handle (F) and, when not in use, can be hanged to the support (5) present on main housing.

FUNZIONING

Preparation:

- Examine the unit before the first use in order to check for possible damage.
- Before using the device, proceed with the cleaning operations as described in “Cleaning” section. Place on a stable surface, near to an electrical socket.
- Check that the filter is properly inserted in its seat (2).
- Unwind the power supply cord and plug in.

Mounting:

- Prepare the nebulizer inserting the medicine in it, and mounting it.
- Connect one side of the tube (8) to the air outlet (3).
- Connect the other side of the tube (E) to the lower part of nebulizer.
- Take a mask, or the mouthpiece and mount to the air outlet (C) of nebulizer.

Usage:

- Press the power switch (1) to switch on the appliance.
- Breathe in the mask until the medication in the nebulizer has been completely consumed.

WARNING: This device rated for intermittent use 30 minutes ON/ 30 minutes OFF.

After treatment:

- Turn off the appliance by pressing its switch (1) to position 0
- Unplug the appliance from the power socket.
- Detach all the accessories and clean them as indicated in the following paragraph.

MANTAINANCE and CLEANING

Aerosoltherapy System does not need any special care, you just should:

- avoid knocking it or dropping it;
- store it in a suitable place, away from excessive heat, direct sunlight, damp and dust;
- clean it with a slightly damp cloth and/or mild detergent (do not use benzene, thinners or similar substances), and dry it carefully and thoroughly;
- never soak or immerse the appliance in water;
- never use the appliance continuously for longer than 30 minutes.

CLEANING

After each treatment, it is suggested to clean the accessories in the proper way.

Nebulizer (6):

- clean by washing in liquid soap and water, then rinse.
- disinfect by boiling in water (max. 8 minutes) or in an autoclave (max. 121°C for 8 minutes).

Mouthpiece (7) and masks (9 and 10):

- clean by washing in liquid soap and water, then rinse.
- disinfect using cold disinfecting liquids.

Dry all parts using a clean cloth.

FILTER

Replace the air filter approximately after 30 hours of use, or when it appears dirty.

- open the filter compartment (2).
- remove the old filter and insert a new one. replace the cover on the filter compartment (2).

TROUBLESHOOTING

Problems	Solutions
The appliance does not operate.	Check if the power plug is properly inserted. Respect the usage time 30 minutes ON/ 30 minutes OFF.
The medication does not nebulizer properly.	Check if the nebulizer and the tube have been properly assembled. Check if the medicine is finished in the nebulizer container.
The device is noisy.	Check that the filter is correctly positioned into the compartment (2).

SPECIFICATIONS

- Compressor: Oilless and maintenance-free piston pump
- Operation Pressure Range: 12-14.5 Psi
- Airflow range: 8-10 l/min
- Medication capacity: 6-8ml
- Particle size: Respirable fraction 0.5 to 5 μ m, 0.5 μ m 80%
- Mass Median Aerodynamic Diameter (MMAD): 3-5 μ m
- Average nebulization rate: > 0.3 ml/min
- Noise level: < 60 dBA
- Power supply: AC 230V ~ 50Hz
- Power consumption: 65W

WASTE OF ELECTRIC AND ELECTRONIC EQUIPMENT

The product is made of non-biodegradable and potentially polluting substances if not properly disposed of; other parts can be recycled. It's our duty to contribute to the ecological health of the environment following the correct procedures for disposal. The crossed out wheeled bin symbol indicates the product complies with the requirements of the new directives introduced to protect the environment (2002/96/EC, 2003/108/EC, 2002/95/EC, 2012/19/EC) and must be properly disposed at the end of its lifetime. If you need further information, ask for the dedicated waste disposal areas at your place of residence. Who does not dispose of the product as specified in this section shall be liable according to the law.

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To find the nearest service center, please call the telephone number 0541 694246, fax number 0541 756430 or contact us at assistentzatecnica@trevidea.it asking for the service centers supervisor.

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Appliance is conform to the pertinent European regulation and is therefore marked with the CE mark.

C € 0598

www.g3ferrari.it

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**TREVIDEA S.R.L. – Strada Consolare Rimini – San Marino n. 62 – 47924 – RIMINI
(RN) – ITALY**

Foshan HongFeng Co., Ltd.

No. 4-2, Leqiang Road, Leping Town, Sanshui District, Foshan City, Guandong Province, P.R. China

MedNet EC-REP GmbH, Borkstrasse 10, 48163 Münster, Germany.

MADE IN CHINA

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TREVIDEA S.r.l. si scusa inoltre per eventuali errori di stampa.